




HOW TO USE ?



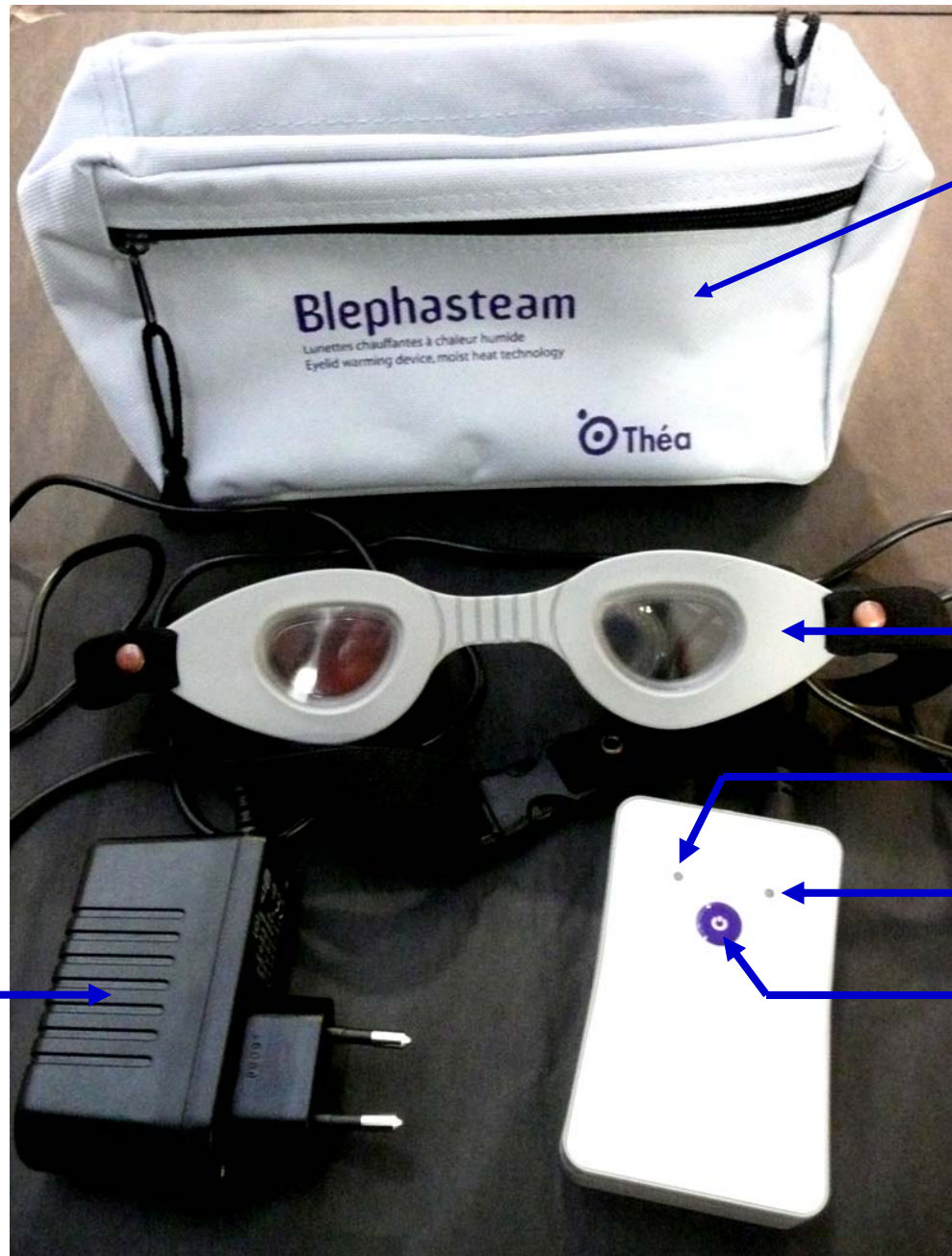

Storage pouch

Eye-warming goggles

Red light

Green light

On/Off button

Electric plug



Wash and dry your hands before use




Plug the power adapter into a socket



You should hear a beep.



keep the power adapter plugged in throughout the complete session.

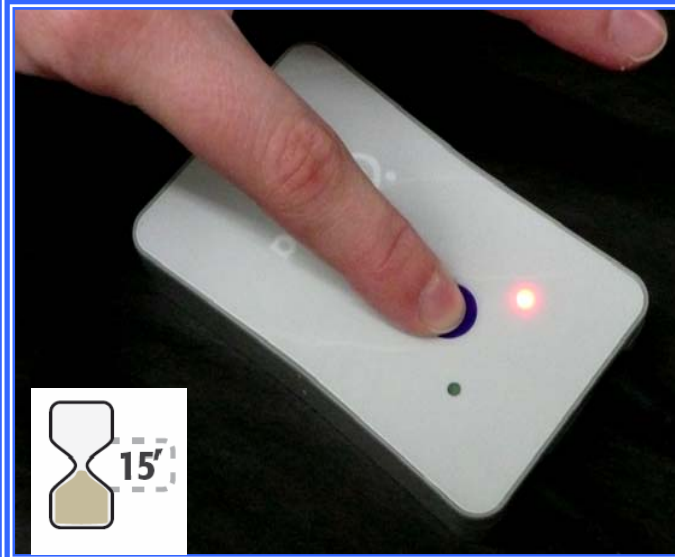


Press and release the middle blue button: the red light should turn on. **The device is now preheating.**

Once the preheating step is complete (15 min), the green light comes on and the device is now ready for use.



Preheating takes 15 minutes.

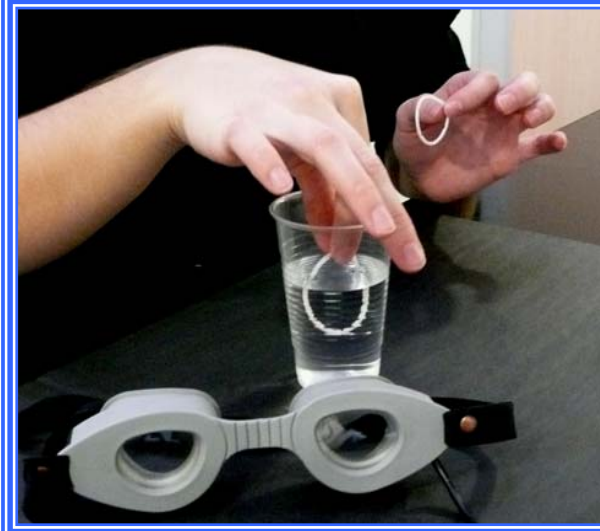

Moisten two BLEPHASTEAM® rings preferably with mineral water or saline solution.



Blephasteam goggles have to be used with pre-wetted rings



Blephasteam[®]


Eyelid warming device, moist heat technology



Place these wet rings inside the device.

Ensure that the rings are placed flat against the lens


Fit the device on your face
So it is comfortable over your eyes.

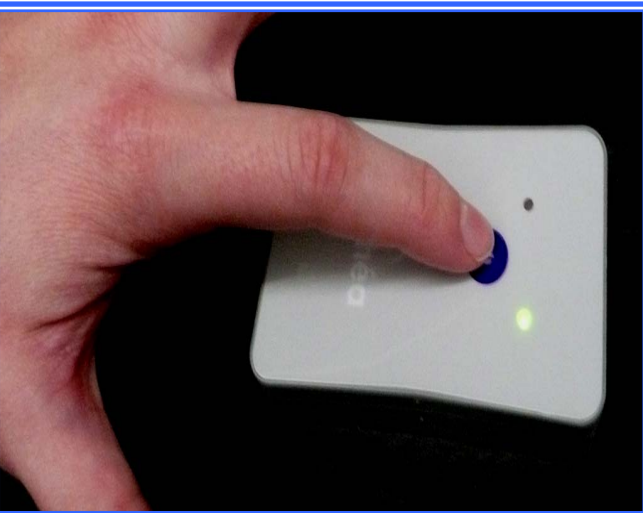

To start the session press and release the middle blue button: **the green light should now start to flash.**



The session will last 10 minutes

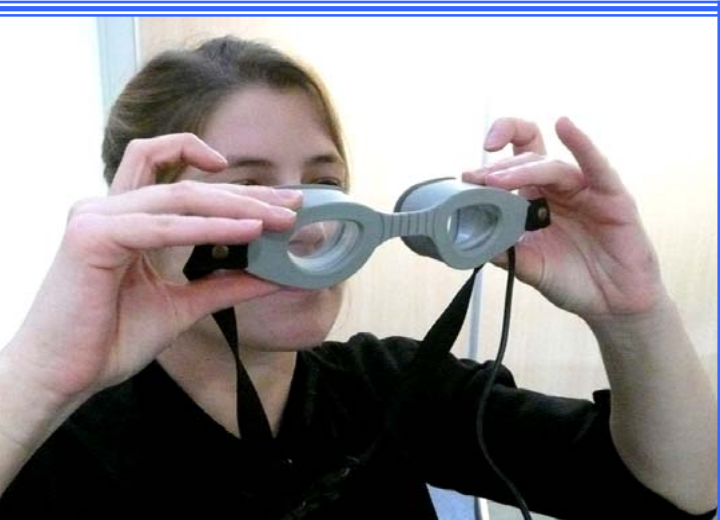

Take the device off your head




Throw the rings away



Unplug the power adapter



The rings are only for 1 session and must be replaced for a new session

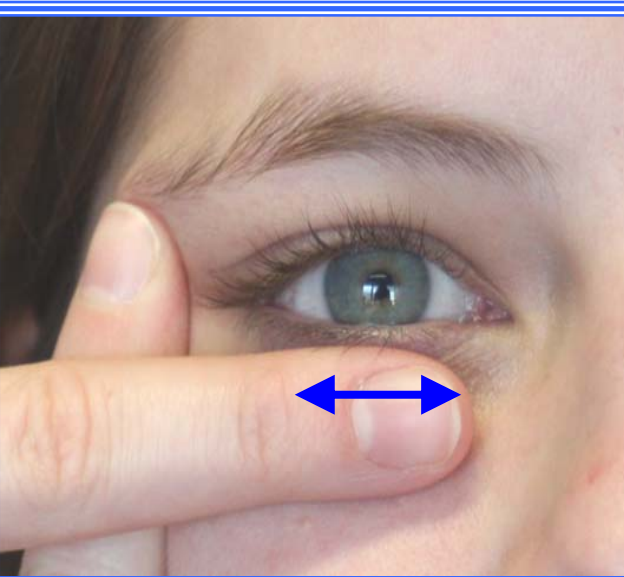

Massage your eyelids

Clean your eyelids with a pad such as BLEPHACLEAN®.



Blephasteam®

Eyelid warming device, moist heat technology



Clean the device
with the provided dry blue cloth
or an alcohol-soaked wipe

Put it into the storage pouch

Wash your hands
after use



The device is only for individual use and should not be shared

